

Installation/Mounting instructions for HealthLink's GBG Foaming Instant Hand Sanitizer Wall Bracket (HealthLink #7788)

Items Required in addition to the wall bracket:

Hammer

Drill

Phillips head screw driver

4 #8 self tapping 1" long Phillips pan head screws (or similar) and 4 plastic wall anchors

Pencil

Mounting Instructions

Remove bracket and drip cup from plastic bag

Locate appropriate smooth wall surface for mounting the bracket

Place mounting bracket at appropriate height on wall and, with a pencil, mark the mounting holes on the wall

Drill holes in the mounting surface at marked points and secure plastic wall anchors in the wall

Place drip cup behind the mounting bracket (see figure 1), snap in to place, and position mounting bracket on wall where wall anchors have been secured.

Using a Phillips head screw driver, secure bracket and drip cup to wall anchors using Phillips pan head screws.

Figure 1

