

surface disinfection



For Use On Hard Surfaces Where Control Of Cross Contamination Is Required

Using EZ-Kill® Wipes in a 2 Step Process



Step 1: Clean Before Disinfecting

A. Pull out an EZ-KILL wipe.



B. Clean surface with wipe to remove debris and bioburden.



C. Discard wipe after cleaning.



Step 2: Disinfect After Cleaning

A. Pull out another EZ-KILL wipe.



B. Wipe evenly and thoroughly to disinfect the surface.



C. Discard wipe.



Use EZ-KILL® wipes to clean, disinfect and decontaminate hard surfaces where control of cross contamination is required:

- hospital
- intensive care units
- ambulances
- newborn nursery
- orthopedics
- patient rooms
- medical offices
- dental offices
- veterinary facilities
- emergency medical settings
- exam rooms
- kitchens
- nursing facilities
- physician offices
- respiratory therapy
- other similar areas



Item #	Product Description	Packaging
7105	Disinfecting Wipe 10" x 10" sheet size	65 wipes / tub 12 tubs / cs
7110	Disinfecting Wipe 6" x 6.75" sheet size	160 wipes / tub 12 tubs / cs