

PRODUCT INFORMATION AND QUALITY CONTROL SHEET

MacCONKEY AGAR

I. INTENDED USE

MacConkey Agar is a selective and differential medium for the isolation and differentiation of gram negative enteric bacilli.

II. SUMMARY AND EXPLANATION

Since this bile salt, neutral red, and lactose agar was described by MacConkey in 1905, its formulation has been modified many times. The current formulation was designed to exhibit better differentiation of lactose fermenting versus non-lactose fermenting organisms, enhance the growth of enteric pathogens, and improve the inhibition of swarming *Proteus* species.

III. PRINCIPLES OF THE PROCEDURE

The selective property of this medium is due to the addition of bile salts and crystal violet, which inhibit the growth of gram positive organisms. Differentiation of lactose fermenting versus non-lactose fermenting is achieved with the indicator neutral red. Colonies will appear either colorless or pink to red based on the organisms ability to ferment this carbohydrate.

IV. TYPICAL FORMULA AND APPEARANCE

Appearance =pinkish-purple, slightly opalescent
(Approximate formula* per liter of processed water)

Pancreatic Digest of Gelatin	17.0g
Pancreatic Digest of Casein	1.5
Peptic Digest of Animal Tissue	1.5
Lactose	10.0
Bile Salts	1.5
Sodium Chloride	5.0
Neutral Red	0.03
Crystal Violet	0.001
Agar	13.5

*adjusted and/or supplemented to meet performance criteria.

V. PRECAUTIONS

This product is for IN VITRO diagnostic use only. Culture specimens may contain microorganisms, which can be potentially infectious to the user. Strict adherence to aseptic techniques and established precautions against microbiological hazards should be followed throughout the procedure. Carefully dispose of all items, which contact patient specimens or isolated bacteria.

VI. STORAGE/SHELF LIFE

Plated media should be stored at 2-8°C (36-46°F), media side up, in the unopened or resealed package protected from light. DO NOT FREEZE OR EXPOSE TO HIGH TEMPERATURES. Allow unopened plates to warm to room temperature prior to inoculation. Prior to and during inoculation procedures, plates should be handled in a manner that minimizes product exposure to the environment. Product which has exceeded the assigned expiration date noted on the label should not be used. Do not use plates that exhibit evidence of drying, cracking, discoloration, microbial contamination or any other signs of deterioration. The presence of excessive condensate may indicate plates that have been damaged by exposure to temperature extremes.

VII. SPECIMEN COLLECTION

The quality of culture results depends primarily on the adequacy and condition of the specimen submitted for examination. Proper specimen collection techniques must be followed to ensure the most accurate culture results. Sterile swabs and collection containers should be used. Plates should be inoculated promptly after specimen collection. If a delay in inoculation is unavoidable, transport medium should be employed. Specimens should be collected prior to the initiation of antimicrobial therapy.

Detailed information on proper specimen collection may be obtained from microbiology reference materials.

VIII. MATERIALS PROVIDED

MacConkey Agar Plates (10 each per sleeve)

IX. MATERIALS REQUIRED BUT NOT PROVIDED

Incubator maintaining 33-37°C.

Ancillary culture media, reagents and laboratory equipment as required.

X. PROCEDURE

Inoculate the specimen as soon as possible after it is received in the laboratory. The streak plate method is used primarily to isolate pure cultures from specimens containing mixed flora. If material is being cultured directly from a swab, roll the swab over a small area of the plate surface at the edge (approximately 1/4 to 1/3 of the plate); then streak in a zig-zag fashion with a sterile loop from this inoculated area in two sections to cover the entire agar surface; flaming or flipping loop between sections. Avoid applying excessive pressure to the agar surface during inoculation to prevent gouging and splitting of the agar media. Incubate plates media side up at 33-37°C for 18-24 hours. (Note: Agar surfaces should be smooth and moist but free of excessive moisture, which could cause confluent growth patterns.)

A non-selective medium, such as Tryptic Soy Agar with 5% Sheep Blood, should also be considered for inoculation to increase the chance of recovering gram negative bacteria present in low numbers and to reflect other bacterial flora present in the specimen.

XI. EXPECTED RESULTS

NCCLS CONTROL ORGANISMS (ATCC STRAINS)

<i>Escherichia coli</i> (ATCC 25922)	Growth, pink colonies
<i>Proteus mirabilis</i> (ATCC 12453)	Growth, colorless colonies, inhibition of swarming
<i>Salmonella typhimurium</i> (ATCC 14028)	Growth, colorless colonies
<i>Enterococcus faecalis</i> (ATCC 29212)	Inhibition (partial)

XII. LABORATORY RESULTS

This medium is intended to be used as a primary isolation medium. Presumptive identification of organisms may be made on the basis of typical organism morphology and Gram stain. Definitive identification of organisms requires further testing. Additional biochemical information may be obtained from reference microbiology texts.^{1,2,3}

XIII. LIMITATIONS

The ability to detect microorganisms by culture techniques can be affected by the following factors: improper specimen collection, storage and inoculation, initiation of antiinfective therapy prior to specimen collection, improper culture incubation temperatures and atmospheres, improper length of culture incubation, and improper storage and handling of culture media.

XIV. REFERENCES

1. Finegold, S.M. and W.S. Martin. 1982. Bailey and Scott's Diagnostic Microbiology, 6th ed. C.V. Mosby Company, St. Louis.
2. Koneman, E.S., S.D.Allen, V.R. Dowell, Jr. and H. M.

Sommers. 1983. Color Atlas and Textbook of Microbiology, 2nd ed. J.B. Lippincot Company, Philadelphia.

3. Lennette, E.H., ed. 1985. Manual of Clinical Microbiology, 4th ed. American Society for Microbiology, Washington, D.C.

USER QUALITY ASSURANCE/ QUALITY CONTROL PROCEDURES AND INFORMATION

HealthLink recommends that the following quality assurance and quality control procedures be performed on each batch of product.

I. QUALITY ASSURANCE

The following quality assurance procedures must be performed to assure the product will perform according to its intended use within the assigned expiry date:

1. Daily, document that product storage refrigerator maintains temperature within the recommended range: 2-8°C.
2. Daily, document that laboratory incubator maintains temperature within the recommended range: 22-35°C.

II. QUALITY CONTROL

The following incoming inspection procedure must be performed for each batch (batch = same lot, same shipment) of culture media received in the laboratory :

Inspect plates according to instructions contained in Section VI "STORAGE/SHELF LIFE"

Note: Notify Technical Service immediately if media does not meet the inspection criteria.

TECHNICAL SERVICE

HealthLink provides a toll free technical service line (1-800-638-2625) to assist with product usage. To have technical questions answered; please call between the hours of 9:00 am to 5:00 pm EST.

**HealthLink
3611 St. Johns Bluff Rd. So. Ste 1
Jacksonville, FL 32224**

1-800-638-2625

June, 1999

Product No. 1087 Rev. No. 01